

# Instructions For Use - Reusable Sterile Surgical Drapes and Gowns

## Product Description

Reusable Sterile Surgical Drapes and Gowns are composed recyclable laminate fabrics which are manufactured by bonding a breathable membrane between 2 layers of textiles and coating the resulting product with a fluid proof solution for guaranteed waterproof protection. The membrane enables the laminated fabric to be waterproof from the outside while at the same time allowing perspiration to pass through from the inside. Designed to create a personal protective barrier between health care personnel during surgical procedures to protect both the patient and health care personnel from the transfer of microorganisms, body fluids, and particulate matter. Reusable Sterile Surgical Drapes and Gowns a sterilised using Steam Sterilisation.



### Caution

This product should only be used by a trained professional only.

## Preparation for Use

### Packaging Inspection



Do not use the product if the pack is damaged or open.

Before opening any medical pack, visually inspect the packaging. Ensure that the packaging is intact, free from tears, punctures, or any signs of damage.

### Label inspection

Inspect manufacturing date, expiry date, batch number, and reference number are printed on the product label.

Do not use any product past its expiration date.

## Intended use and Collection

Reusable Sterile Surgical Drapes and Gowns are intended to be used on an individual patient only during a single surgical procedure and then stored using the agreed method of place in used garments into laundry bags and bins for collection by Priontex. Reusable Drapes and Gown are intended to be processed **ONLY** by Priontex using the Priontex facilities and validated methods.

## Handling and Storage

Products are sterile and shall be stored in its original packaging, in a cool dry place.

The product shall be used as soon as the package is opened.

Do not expose to heat or sunshine directly.

## Instructions for use

1. Reusable Sterile Surgical Drapes and Gowns are ready for use.
2. Open Packaging Carefully -Use caution when opening medical pack packaging to avoid damaging the contents or packaging.
3. Prior to use, the outer box must be left outside of the operation room. The product shall be opened in aseptic conditions, and then removed from the original pack in sterile and aseptic conditions.
4. To open the sterilization package, use a thumb and index finger to hold plastic side of pack and a thumb and index finger to pull back paper side of pack. For a clean peel, open the package slowly.



Do not re-use



Do not use is packaging is damaged



Keep Away from direct Sunlight



Single sterile barrier system with protective packaging inside



Do not resterilise



Medical Device



Keep Dry



Product has been sterilised using Steam Sterilisation



Priontex SA (PTY) Ltd  
Corporate Park North 313 Roan Crescent  
Erf 493 Off R101 Old Pretoria Road  
Randjespark Ext. 121 1683  
South Africa  
Tel: (+27) 011 237 5900  
Email: info@priontex.com  
www.priontex.com

