

# Instructions For Use - Single Use Sterile Surgical Drapes and Gowns

## Product Description

Designed to create a personal protective barrier between health care personnel during surgical procedures to protect both the patient and health care personnel from the transfer of microorganisms, body fluids, and particulate matter. Single Use Sterile Surgical Drapes and Gowns are sterilised using Ethylene Oxide.



### Caution

This product should only be used by a trained professional only.

## Preparation for Use

### Preparation for Use

#### Packaging Inspection



Do not use the product if the pack is damaged or open.

Before opening any medical pack, visually inspect the packaging. Ensure that the packaging is intact, free from tears, punctures, or any signs of damage.

#### Label inspection

Inspect manufacturing date, expiry date, batch number, and reference number are printed on the product label.

Do not use any product past its expiration date.

## Intended use and Disposal

Single Use Sterile Surgical Drapes and Gowns are intended to be used on an individual patient during a single surgical procedure and then disposed of using the hospital's Medical Waste Handling Procedure. It is not intended to be reprocessed and used again.

## Handling and Storage

Products are sterile and shall be stored in its original carton box, in a cool dry place.

The product shall be used as soon as the package is opened.

Do not expose to heat or sunshine directly.

## Instructions for use

1. Single Use Sterile Surgical Drapes and Gowns are ready for use.
2. Open Packaging Carefully -Use caution when opening medical pack packaging to avoid damaging the contents or packaging.
3. Prior to use, the outer box must be left outside of the operation room. The product shall be opened in aseptic conditions, and then removed from the original pack in sterile and aseptic conditions.
4. To open the sterilization package, use a thumb and index finger to hold the plastic side of the pack and a thumb and index finger to pull back the paper side of the pack. Always open the pack from the direction of the arrow. For a clean peel, open the package slowly.



Do not re-use



Do not resterilise



Latex Free



Medical Device



Keep Away from direct Sunlight



Keep Dry



Single sterile barrier system with protective packaging inside



Product has been sterilised using Ethylene Oxide



Priontex SA (PTY) Ltd  
Corporate Park North 313 Roan Crescent  
Erf 493 Off R101 Old Pretoria Road  
Randjespark Ext. 121 1683  
South Africa  
Tel: (+27) 011 237 5900  
Email: [info@priontex.com](mailto:info@priontex.com)  
[www.priontex.com](http://www.priontex.com)

**PrionGuard**