

Instructions For Use - Custom Operating Packs (COP)

Product Description

Custom Operating Packs are customised to fit specific user requirements and are a key to free up clinical time to focus on quality patient care and reduce costs. Custom Operating Packs are sterilised using Ethylene Oxide.

General

Minor Procedure Trays
Catheter Packs

Orthopaedics

Knee Packs
Hip Packs
Lower Extremity Packs
Foot and Hand Pack

Obstetrics & Gynae

Caesarean Section Packs
D&C Packs
Obstetrics Packs
Lithotomy Packs

Ophthalmology

Cataract Packs
Intravitreal Packs
Eye Cleansing Trays

ENT

Ear Packs
T&A Full Packs
Myringotomy Packs
Nasal Packs
Thyroid Packs



Caution

This product should be used by a trained professional only.

Preparation for Use

Packaging Inspection

 Do not use if the packaging is damaged.

Before opening any medical pack, visually inspect the packaging. Ensure that the packaging is intact, free from tears, punctures, or any signs of damage.

Label inspection

Inspect manufacturing date, expiry date, batch number, and reference number are printed on the product label. Do not use any product past its expiration date.

Intended use and Disposal

Custom Operating Packs are intended to be used on an individual patient during a single surgical procedure and then disposed of using the hospital's Medical Waste Handling Procedure. It is not intended to be reprocessed and used again.

Handling and Storage

Products are sterile and shall be stored in their original carton box, in a cool dry place.

The product shall be used as soon as the package is opened.

Do not expose to heat or sunshine directly.

Instructions for use

1. Ensure opening in a sterile environment as required using the aseptic techniques.
2. Open primary packaging by either peeling back paper liner or tearing along indicated perforated line.
3. Where a table cover is included in the pack open the fold away from the healthcare user first to maintain a sterile field.
4. Open primary packaging by either peeling back paper liner or tearing along indicated perforated line.
5. Where a table cover is included in the pack open the fold away from the healthcare user first to maintain a sterile field
6. Remove the contents and place within the sterile field as appropriate.
7. Any secondary packaging within the primary pack should be opened follow same opening technique as described above.



Do not re-use



Do not resterilise



Latex Free



Medical Device



Keep Away from direct Sunlight



Keep Dry



Single sterile barrier system with protective packaging inside



Product has been sterilised using Ethylene Oxide



Priontex SA (PTY) Ltd
Corporate Park North 313 Roan Crescent
Erf 493 Off R101 Old Pretoria Road
Randjespark Ext. 121 1683
South Africa
Tel: (+27) 011 237 5900
Email: info@priontex.com
www.priontex.com

